

Anne Arundel Diagnostics Imaging

How to easily use our web based PACS

1. To log in, enter your USER ID then your PASSWORD. Set the CAPS Lock. Remember, your ID and PASSWORD are case sensitive
2. To locate a patient type the last name in the box under Patient Name.
3. Find the exam by double clicking on the correct patient.
4. Double click on the desired exam to display it.
5. Window and Level the images by clicking and dragging the left mouse left/right to window and up/down to level.
6. Click on the right mouse for these functions; magnify, measure distance, image flip, rotate, save and print.
7. To close an exam click on either the CLOSE or BACK button in the upper left corner of the monitor.
8. To log out click on the “X” button in the right upper corner of the window.

NEED HELP?

Call the Help Desk at 443-481-5202